

neodisher® MediClean forte Cleaning, Disinfecting and Drying Equipment

Liquid detergent concentrate for the manual and machine cleaning of breathing masks, respiratory protection equipment, diving equipment and chemical protection suits before disinfection.



D-98897-2013

Benefits

Universal cleaner for dual use

neodisher® MediClean forte is an alkaline enzymatic universal cleaning concentrate. You clean the personal protective equipment with the cleaning solution manually in an immersion bath with a cloth and sponge before disinfection. For chemical protection suits you can also use a brush. Thanks to the special alkaline enzymatic formulation, you can also use neodisher® MediClean forte for machine cleaning.

Product advantages at a glance

- First class cleaning performance
 - Very good material compatibility (exception: Dräger FPS 7000 RP, Dräger Panorama Nova RP)
 - User-friendly formula
 - Use for manual and machine cleaning
 - Economical due to low dosage (depending on degree of contamination: 10 ml/l - 30 ml/l (1.0 % - 3.0 %))
 - No markings: No hazardous material; No dangerous goods
-

Information for use

- When cleaning in an immersion bath, respiratory protection equipment and accessories must be completely wetted with the application solution during the entire immersion time.
- Then rinse with running water of at least potable water quality.
- Cold water up to max. 40 °C
- The respective Dräger instructions for use for respirators, respiratory protection equipment, chemical protection suits, diving equipment, etc., as well as the instructions for use from the cleaning agent manufacturer must be followed!

Accessories



D-23495-2020

neomatik® ZMW 1 compact

Compact injector mixing equipment for the preparation of ready-to-use solutions of cleaning and disinfecting agents. The dosing is done by an injector according to the Venturi principle.



D-23496-2020

neomatik® ZMW 4 compact

Compact injector mixing equipment for the production of a maximum of four different ready-to-use solutions of cleaning and disinfecting agents. The selection is made via a rotary knob. The dosing is done by an injector according to the Venturi principle.

Related Products



D-98828-2013

neoform® K plus

Liquid disinfectant cleaning concentrate for the disinfection and cleaning of breathing masks, respiratory protection equipment, diving equipment, chemical protection suits and surfaces (e.g. drying cabinets, work surfaces, floors).

Related Products



Incidin™ Alcohol Wipe

The absence of active residues combined with broad efficacy spectrum in short contact times make Incidin™ Alcohol Wipe ideal for use on alcohol resistance surfaces, delivering disinfection results and compliance levels mandatory to restrict the spread of pathogens.



Skin disinfection

Tough against bacteria – but soft on your skin. The gentle cleaning, disinfecting and care products protect the skin even if used frequently.

ST-7476-2008

Technical Data

Mechanical and manual cleaning of personal protective equipment

Manual cleaning of breathing apparatus and accessories in an immersion bath	Concentration ¹ :	10 ml/l - 30 ml/l (1.0 % - 3.0 %) ²
	Temperature:	Cold water up to max. +40 °C
	Cleaning time:	15 - 30 minutes
Machine cleaning of certain personal protective equipment (release according to manufacturer's instructions), breathing apparatus and accessories in washer-disinfectors and/or washing tumbling machines (specification of special programs according to manufacturer's instructions)	Concentration ¹ :	10 ml/l (1 %)
	Temperature:	+55 °C
	Cleaning time:	2 minutes

¹ See separate dosing table 5 l - 50 l (on requirement)

² The dosing amount depends on the degree of contamination.

Information for use:

Example: 10 litres = 1 % application solution = 100 ml product concentrate + 9.9 litres water. Add the amount of product concentrate corresponding to the water volume, stir briefly. Solution is ready for use immediately. The solution can be used for one working day with normal contamination. In case of visible soiling, please prepare fresh cleaning solution.

Ordering Information

Product	Countries	Packaging	Order number
neodisher® MediClean forte 5L	AT; CH; DE; LI	5 litre compact canister	37 06 140
neodisher® MediClean forte 5L	GB; IE; MT	5 litre compact canister	37 08 062
neodisher® MediClean forte 5L	BE; FR; LU; NL	5 litre compact canister	37 06 063
neodisher® MediClean forte 5L	CZ; HU; PL	5 litre compact canister	37 06 135
neodisher® MediClean forte 5L	HR; SI; SK	5 litre compact canister	37 06 141
neodisher® MediClean forte 5L	ES; GR; IT; PT	5 litre compact canister	37 06 136
neodisher® MediClean forte 5L	DK; FI; NO; SE	5 litre compact canister	37 06 137
neodisher® MediClean forte 5L	KZ; RU; UA	5 litre compact canister	37 06 138
neodisher® MediClean forte 5L	EE; LT; LV	5 litre compact canister	37 06 142
neodisher® MediClean forte 5L	AU; NZ	5 litre compact canister	37 08 064
neodisher® MediClean forte 5L	AE; SA; ZA	5 litre compact canister	37 08 065

Accessories

	Order number
Canister wrench for 5 - 20 l canisters	37 06 161
Angled drain cock for 5 and 10 l canisters	37 06 162
Measuring cup 500 ml with scale	37 06 163
neomatik® ZMW 1 compact	69 26 963
neomatik® ZMW 4 compact	69 26 964

neodisher®, neoform® and neomatik® are registered trademarks of Dr. WEIGERT GmbH & Co.KG

Incidin™ is a registered trademark of Ecolab.

Notes

Not all products, features, or services are for sale in all countries.
Mentioned Trademarks are only registered in certain countries and not necessarily in the country in which this material is released. Go to www.draeger.com/trademarks to find the current status.

CORPORATE HEADQUARTERS

Drägerwerk AG & Co. KGaA
Moislinger Allee 53–55
23558 Lübeck, Germany
www.draeger.com

Manufacturer:

Chemische Fabrik
Dr. Weigert GmbH & Co. KG
Mühlenhagen 85
20539 Hamburg
Germany
Tel.: +49 (0)40 789 60-0

REGION EUROPE

Dräger Safety AG & Co. KGaA
Revalstraße 1
23560 Lübeck, Germany
Tel +49 451 882 0
Fax +49 451 882 2080
info@draeger.com

REGION MIDDLE EAST, AFRICA

Dräger Safety AG & Co. KGaA
Branch Office
P.O. Box 505108
Dubai, United Arab Emirates
Tel +971 4 4294 600
Fax +971 4 4294 699
contactuae@draeger.com

REGION ASIA PACIFIC

Draeger Singapore Pte. Ltd.
61 Science Park Road
The Galen #04-01
Singapore 117525
Tel: +65 6872 9288
Fax: +65 6259 0398
asia.pacific@draeger.com

REGION CENTRAL AND SOUTH AMERICA

Dräger Indústria e Comércio Ltda.
Al. Pucurui - 51 - Tamboré
06406-100 - Barueri - SP
Tel. +55 (11) 4689-4900
relacionamento@draeger.com

Locate your Regional Sales
Representative at:
www.draeger.com/contact

